

Compat ella[®]

Compat Ella[®] Dual Spike Set

The Compat Ella[®] Dual Spike Set is intended for the simultaneous administration of enteral nutrition and/or hydration via two separately spiked containers and is suitable for use with the Compat Ella[®] feeding pump.

Features at a glance

Dual Spike: For simultaneous feeding and/or hydration via two separately spiked containers.


Built-in Safety: All Compat Ella[®] administration sets feature patented fail-safe **Auto-Anti-Free-Flow (AAFF)** technology. **AAFF** is incorporated into the cassette and is always automatically activated to ensure patient safety.

Safer Connectors: All Compat Ella[®] administration sets include ENFit[®] connectors made according to **ISO 80369-3**. **ENFit[®]** and **ENPlus connectors** are designed to be **IV Incompatible** to promote **patient safety** by helping to avoid accidental misconnections.

Food Safe Materials: All Compat Ella[®] administration sets are **DEHP, BPA and Latex-Free***.

Convenient:

- Nutrition containers: ENPlus
- Enteral feeding tubes: ENFit[®]
- Medication dispensers: ENFit[®]

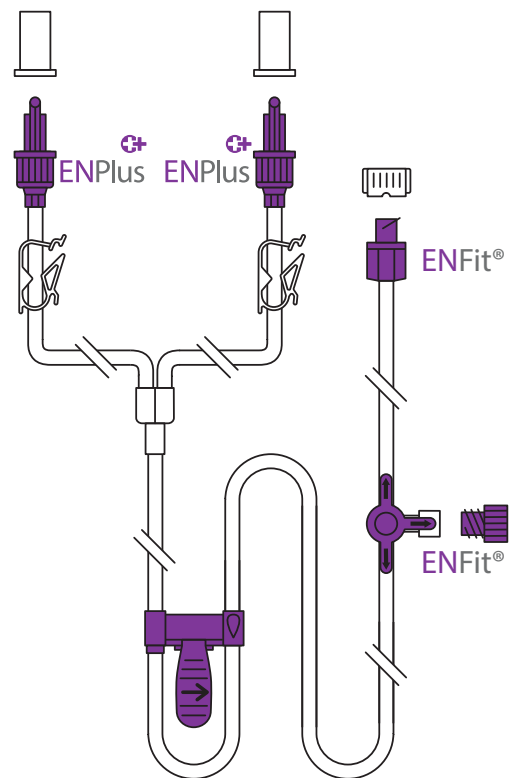


Ordering Information

Product Name	Product Code	Qty/Box
Compat Ella [®] Dual Spike Set	12297288	30

Have you met Compat Ella[®]?

Compat Ella® Dual Spike Set

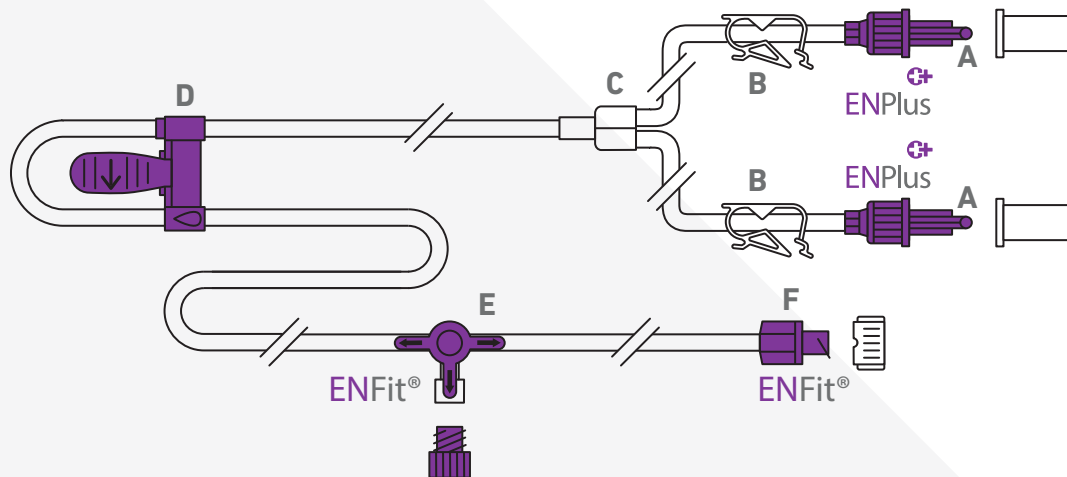

Components

- A ENPlus 'enteral only' proximal end (nutrition connection) with revolving lock and cap
- B Pinch clamp to manually shut off the flow of fluids
- C Y junction
- D Compat Ella® pump cassette with integrated and patented Auto-Anti-Free-Flow (AAFF) safety feature
- E ENFit® 3-way medication port with cap
- F ENFit® distal end with cap (patient side tube connection)

Specifications

Sterile:	Sterilized with Ethylene Oxide.
Packing:	Individually packed in cases of 30 pieces (29.5 x 26 x 19 cm).
Storage:	Store at room temperature. Keep away from sunlight and humidity.
Shelf life:	3 years from production date. Expiry date on the package.
Duration of use:	24 hours. Single Use.
Tube Length:	Distance from nutrition connection to patient side tube connection: 230 cm. Distance from nutrition connection to Y valve: 30 cm. Distance from nutrition connector to pump: 50 cm.
Tube diameters:	Nominal inner diameter: 3.0 mm. Nominal outer diameter: 4.15 mm.
CE Mark:	Complies with the European Medical Device Directive 93/42 EEC. A CE Certification has been issued by an accredited Notified Body: TÜV SÜD (CE0123).
Ecological Footprint:	The 'Grüne Punkt' symbol signals that Nestlé Health Science assumes its producer responsibility and contributes to the financing of the recycling.
Materials:	Tubing: PVC and silicone. All materials DEHP, BPA and Latex-free*. Additional details on request.

To obtain further information please contact your Nestlé Health Science representative or go to www.compatella.com

*The parts do not exceed the maximum concentrations of 0.1% total DEHP (Di (2-Ethyl Hexyl) Phthalate) content. None was intentionally added during the manufacturing process.

*Unless stated otherwise, all trademarks are owned by Société des Produits Nestlé S.A or used with permission. Legal Manufacturer: Cedic Srl. Made in Italy.

NH5MD00404 Rev.1 12DEC2017