

Compat ella®

Compat Ella® Spike Set

The Compat Ella® Spike Set is intended for administration of enteral nutrition, fluids or medication and is suitable for use with the Compat Ella® feeding pump.

Features at a glance

The Compat Ella® Spike Set features an **ENPlus** nutrition connector, an **ENFit® 3-way** medication port and an **ENFit®** distal end (patient side tube connection).


Built-in Safety: All Compat Ella® administration sets feature patented fail-safe **Auto-Anti-Free-Flow (AAFF)** technology. **AAFF** is incorporated into the cassette and is always automatically activated to ensure patient safety.

Safer Connectors: All Compat Ella® administration sets include **ENFit®** connectors made according to **ISO 80369-3**. **ENFit®** and **ENPlus** connectors are designed to be **IV Incompatible** to promote **patient safety** by helping to avoid accidental misconnections.

Food Safe Materials: All Compat Ella® administration sets are **DEHP, BPA and Latex-Free***.

Compatibility:

- Nutrition containers: ENPlus
- Enteral feeding tubes: ENFit®
- Medication dispensers: ENFit®

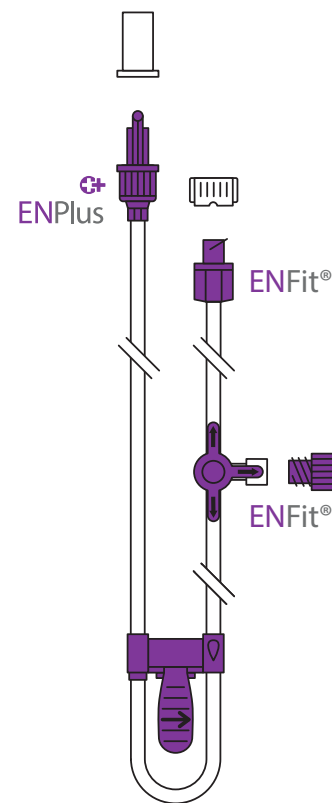


Ordering Information

Product Name	Product Code	Qty/Box
Compat Ella® Spike Set	12298203	30

Have you met Compat Ella®?

Compat Ella® Spike Set

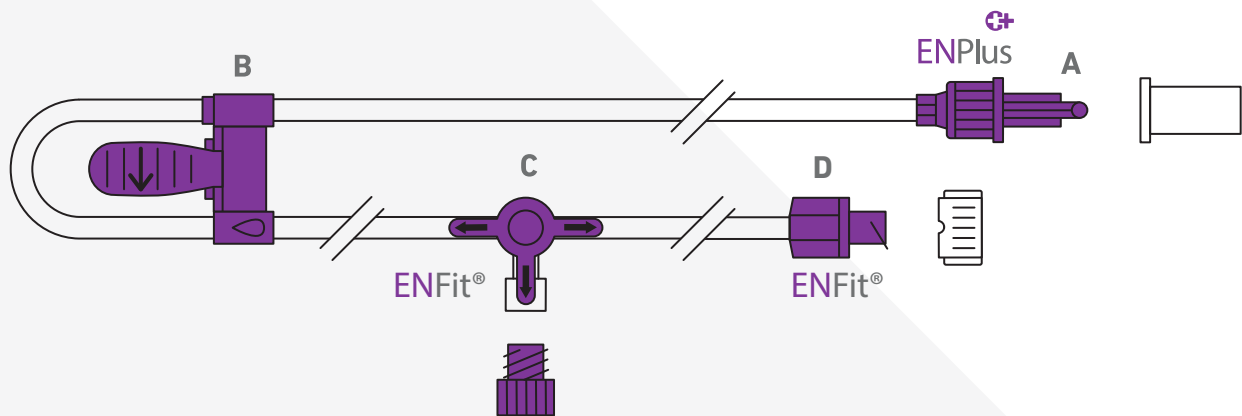

Components

- A ENPlus 'enteral only' proximal end (nutrition connection) with revolving lock and cap
- B Compat Ella® pump cassette with integrated and patented Auto-Anti-Free-Flow (AAFF) safety feature
- C ENFit® 3-way medication port with cap
- D ENFit® distal end with cap (patient side tube connection)

Specifications

- Sterile:** Sterilized with Ethylene Oxide.
- Packing:** Individually packed in cases of 30 pieces (29.5 x 26 x 19 cm).
- Storage:** Store at room temperature. Keep away from sunlight and humidity.
- Shelf life:** 3 years from production date. Expiry date on the package.
- Duration of use:** 24 hours. Single Use.
- Tube Length:** Distance from nutrition connection to patient side tube connection: 230 cm.
Distance from nutrition connector to pump: 50 cm.
- Tube diameters:** Nominal inner diameter: 3.0 mm. Nominal outer diameter: 4.15 mm.
- CE Mark:** Complies with the European Medical Device Directive 93/42 EEC. A CE Certification has been issued by an accredited Notified Body: TÜV SÜD (CE0123).
- Ecological Footprint:** The 'Grüne Punkt' symbol signals that Nestlé Health Science assumes its producer responsibility and contributes to the financing of the recycling.
- Materials:** Tubing: PVC and silicone. All materials DEHP, BPA and Latex-free*. Additional details on request.

To obtain further information please contact your Nestlé Health Science representative or go to www.compatella.com

*The parts do not exceed the maximum concentrations of 0.1% total DEHP (Di (2-Ethyl Hexyl) Phthalate) content. None was intentionally added during the manufacturing process.

* Unless stated otherwise, all trademarks are owned by Société des Produits Nestlé S.A or used with permission. Legal Manufacturer: Cedic Srl. Made in Italy.

NHSM00405 Rev.1 12DEC2017



Compat ella®